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For Medical Use  
For Dentistry Use

**[Identity Statement]**

Powder-free / Examination / Latex-free / Dentistry / Disposable / Ambidextrous / Textured fingertips / Non-sterile

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**[Precautions]**

Be careful of fingernails, knives and other sharp objects that may cause damage. Do not use if boxage or product is damaged, deformed, has pinholes or other defects. Do not use directly with strong chemicals and solvents. Avoid direct sunlight and artificial beam, and keep away from X-ray equipment after opening. Color variability does not affect quality. Stop use when there is itch, irritation or redness. Be careful of use afterwards. Change gloves for each patient to avoid cross-contamination.

**[Storage]**

1. **Storage:** Be careful of wetting, keep away from high temperature, moisture, direct sunlight, and UV ray such as fluorescent light.  
2. **Expiration:** Expiration is printed on the boxage. (Barcode and EXP DATE on the short side)

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**[Usage]**

Use to protect cross-contamination during examination, caregiving and handling contaminated medical devices, except for surgery.

**[Instructions]**

Remove the top along the perforated line, take out gloves from the opening.

**[Precautions]**

Be careful of fingernails, knives and other sharp objects that may cause damage. Do not use if box or product is damaged, deformed, has pinholes or other defects. Do not use directly with strong chemicals and solvents. Avoid direct sunlight and artificial beam, and keep away from X-ray equipment after opening. Color variability does not affect quality. Stop use when there is itch, irritation or redness. Be careful of use afterwards. Change gloves for each patient to avoid cross-contamination.

**[Storage]**

1. **Storage:** Be careful of wetting, keep away from high temperature, moisture, direct sunlight, and UV ray such as fluorescent light.  
2. **Expiration:** Expiration date is printed on the box. (Barcode and EXP DATE on the short side)

**[Quantity]**

200 gloves by weight/box

Manufactured at Yangzhou Saraguard Medical Supplies Co.,Ltd.  
Standard workshop No.1, Phase I, Comprehensive Bonded Zone, No.9, Yangtze Jiangnan Road, Yangzhou City, Jiangsu Province, China

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